

Get connected quick guide


How to connect your Danalogic hearing aids to the BeMore app.

INTRODUCTION

For information on which smartphones and mobile devices are compatible with which app, visit: www.danalogic.co.uk/connectivity you can start using the app, you must pair your smart devices with your hearing aids. This is easily done – just follow the steps below.

HOW TO PAIR YOUR HEARING AIDS WITH YOUR iOS DEVICE

1. Turn your hearing aids off.
2. Turn on **Bluetooth®** by going to the **Settings** menu on your iOS device. The slider turns green when you activate Bluetooth.
3. Go back to the main **Settings** menu and then, tap on **Accessibility**.
4. Scroll down to **Hearing Devices**.
5. Turn your hearing aids on. Now, your iOS device will be able to detect your hearing aids.
6. Tap on **Pair**. If you have two hearing aids, you will need to accept twice.
7. Go to the App Store and search for BeMore- for your hearing aids.
8. Download the BeMore app, open it and tap **Get Started**.
9. Follow the instructions on the screen to connect to the app.
You are now ready to go.

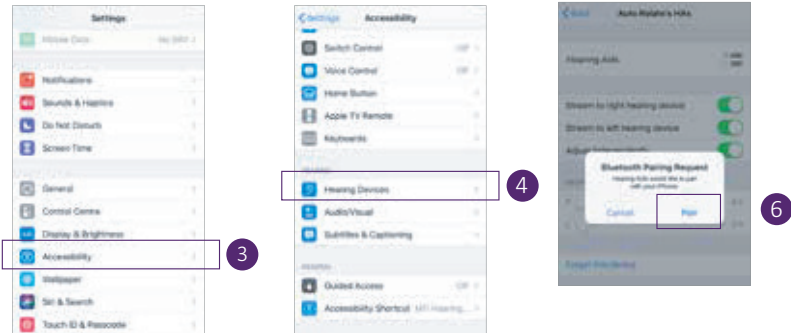

NOTE: These are the steps and how it looks in iOS 13 and newer

danalogic GN



HOW TO PAIR YOUR HEARING AIDS WITH YOUR ANDROID™ DEVICE

1. Go to the Google Play Store, search for BeMore and download the app.
2. Once downloaded, open the app and tap **Get Started**.
3. Follow the instructions on the screen to connect. You are now ready to go.

NOTE: If you have previously paired any hearing aids, you need to unpair them in your smartphone's Bluetooth menu before you pair them again.

IF YOU NEED TO RE-PAIR YOUR iOS DEVICE

1. Start by removing the old pairing by going to the Settings menu.
2. Scroll down and select **Accessibility**.
3. Select **Hearing Devices**.
4. Tap on your hearing aids, scroll down and tap on Forget this device.
5. Turn your iOS device off and on again and re-pair the hearing aids to the device.
If you need more information, contact your hearing care professional.



© 2024 GN Hearing A/S. All rights reserved. Danalogic is a trademark of GN Hearing A/S. Apple, the Apple logo, iPhone, iPad and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries. Android, Google Play and the Google Play logo are trademarks of Google LLC. The Bluetooth word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The Auracast word mark and logos are trademarks owned by Bluetooth SIG, Inc.

UK 10.28 A4 8.2024