

# Ambio battery locks

## Instructions for tamper resistant battery compartment on the Ambio AM562/577/588 models


Insert the right/left indicator into the hearing aid. This is then visible on the bottom of the battery door.

The coloured marker will be set to the left (unlocked position) as a default, use the battery lock tool which comes with the aid to push the coloured marker to the right to lock the battery door.

Alternatively insert the right left indicator into the hearing aid follow the instructions overleaf.

### To lock the battery door:


Open the battery door to the OFF position.

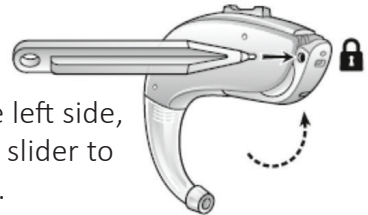

From the left side, push the slider to the right.

### To unlock the battery door (to replace the battery):

Open the battery door to the OFF position.

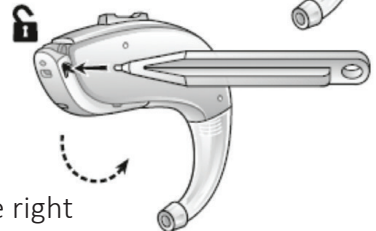

From the right side, push the slider to the left.

Audiologist contact details: