


How to enjoy Hands-Free Calls for iPhone and iPad with Danalogic Extend

Please ensure your device is compatible by checking our website here www.danalogic.co.uk/connectivity

| Function | How to |
|-----------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Answer a call | Use a short press on your hearing aid button or slide to answer on the iPhone/iPad |
| End a call | Press and hold your hearing aid button for a few seconds to end the call. Alternatively, hang up by tapping the red button on your screen |
| Second incoming calls | If you are on a call and you have a second call incoming, you can decline it by using the long press on your hearing aid button or tap 'Decline' on the iPhone or iPad screen |
| Change the volume | Use the up and down volume buttons on the iPhone or iPad |
| Turn off hands-free calls in the iOS hearing aid settings | Triple click the iPhone push button to access the iOS hearing aid settings (you can also get access via the Control Centre). Select 'Input Options' then 'Off'. |

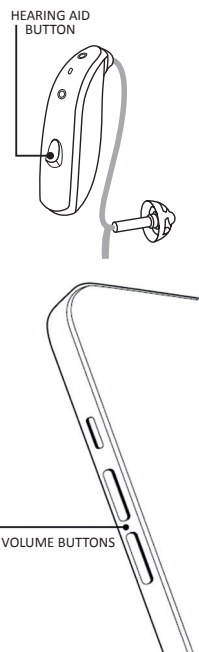

For further support, scan the QR code or visit our patient support portal at: danalogic.co.uk/patient-portal

The right choice for the NHS

danalogic GN

UK 10.24 A 7.2024