

# danalogic Extend BTE LED Light control settings

	Action of software	Turn on	Turn off	In charger		Low battery
				Charging	Fully charged	
LED Light control settings	Disabled					
	Flash Once					
	Continuous Flash					

	Action of software	Change volume	Volume limit	Programme status	DAI	Flight mode
LED Light control settings	Disabled	No light	No light	No light	No light	No light
	Flash Once	1 x when pressed	1 x when reached	Programme 1 Programme 2 Programme 3 Programme 4 1 x when pressed	1 x when activated	1 x when activated (1 long blink, then four double blinks)
	Continuous Flash	1 x when pressed	1 x when reached	Programme 1 Programme 2 Programme 3 Programme 4 1 x when pressed	1 x when activated	1 x when activated (1 long blink, then four double blinks)

Notes:

- Low battery warning takes priority over all other indicators.



For further support, scan the QR code or visit our patient support portal at: [danalogic.co.uk/patient-portal](https://danalogic.co.uk/patient-portal)

The right choice for the NHS

danalogic GN