



# Changing a Danalogic receiver


Insert the tip of the tool into the small slot in the receiver locking mechanism.

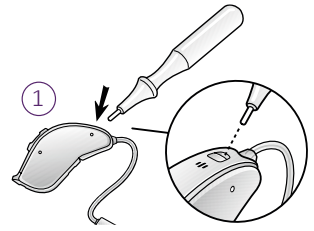


Gently lower the tool at the angle shown, to lift up the clear plastic receiver lock. DO NOT remove the lock from the instrument. Raise it just enough to remove the receiver.

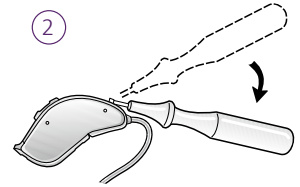


Pull the old receiver straight out to remove it from the hearing instrument.

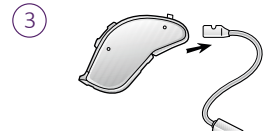

To insert a new receiver, push the new receiver in until it is fully seated.

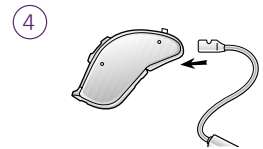

Lock the receiver in place by firmly pushing down the locking mechanism until snugly in place. Attach a dome to the receiver.

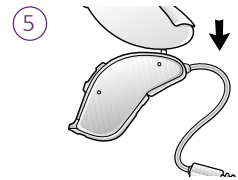

Audiologist contact details: