


Volume control – toggle

Volume control (optional)

The volume control will allow the volume of your hearing devices to be increased or decreased.

- 1 To increase the volume, press the upper end of the toggle.
- 2 To decrease the volume, press the bottom end of the toggle.

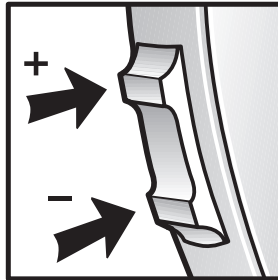

When volume is increased or decreased, a beep signal will be heard for each incremental change. When the upper or lower limits of the volume range are reached, the beep will change tone. During the fitting of your hearing device, your hearing care professional will have chosen an optimal volume setting for you. When switching the hearing device on, the volume will have this optimal setting.

Your hearing device has been set up to beep when the volume is adjusted.

Yes

No

Audiologist contact details: