

Turning on/off and Changing batteries



Turning on and off

Turning the hearing device on

Close the battery drawer completely to switch your hearing device on.

Turning the hearing device off

Open the battery drawer until a click is felt to turn your hearing device off.

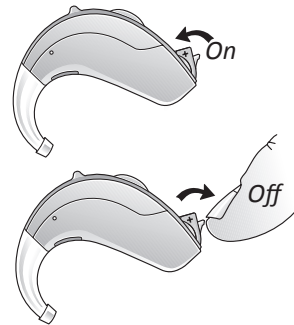
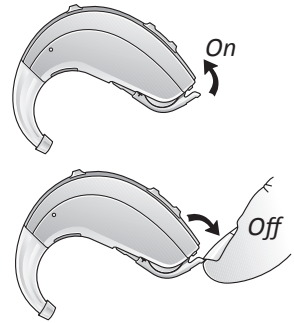


Changing batteries

Battery size and colour:

When the battery power is low

- The hearing device will beep when the battery power gets too low.
- The signal will repeat every 15 minutes until the hearing device automatically turns off.

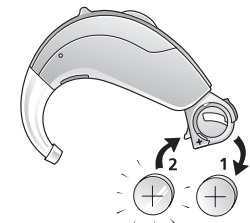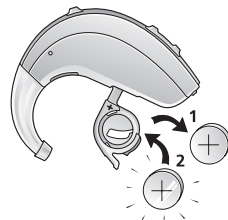

Turning on/off and Changing batteries (continued)

To change the battery


- Remove the sticky label from the new battery.
- Open the battery drawer.
- Remove the dead battery by pulling it upwards.
- Insert the new battery in it's place. Make sure that the flat side (marked +) is facing upwards.

i *To save the battery, make sure you turn your hearing device off when you are not using it.*

Your hearing device has been set up to beep when the battery needs changing.

Yes

No

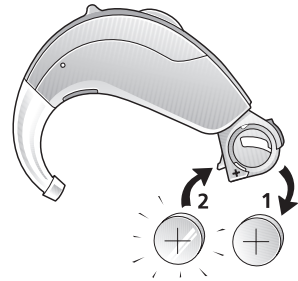

Audiologist contact details: